nailclin

Kalknagelbehandeling

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nailclin

Nail fungus treatment

Please read the user instructions and warnings carefully before use.
Please keep this instructions-for-use leaflet for future reference.

What is Nailclin?

Nailclin is intended for the treatment of nail fungus in adults. Nailclin treats nail fungus with clinically proven effectiveness through application of a patented formula with a practical brush pen. In normal cases, Treatment with Nailclin visibly improves the appearance of the nail within 4 weeks, reducing nail discolouration. Nailclin is based on natural ingredients. The treatment has been designed for daily application and is easy to use. Nailclin is also effective without filing the nail. The liquid saturates the nail plate and restricts fungal growth by creating an unfavourable micro-environment. This way, the body is able to rid itself of the affected nail tissue, replacing it with healthy tissue.. Nailclin is a medical device.

Nailclin consists of:

1. A brush pen containing the Nailclin-formula which is composed of: ethanol (ethyl alcohol), aqua, white thyme oil (10%), lactic acid (3.6%), and eucalyptus citriodora oil (1%).
2. This information leaflet containing important information on the use of Nailclin.

How to use Nailclin:

STEP 1: Clean the affected nail with water

Make sure that the nail is clean and dry; remove any nail polish or lacquers.

STEP 2: Prepare the Nailclin brush pen

Remove the coloured cap and twist the bottom of the pen in the direction of the arrow (see picture) until liquid appears at the brush tip. Upon first use this may take several rotations.

STEP 3: Apply a thin layer of Nailclin to the nail

Cover the entire nail in lotion. Avoid contact with the surrounding skin to avoid skin irritation. Leave the applied layer to dry for one minute. Always reseal the pen with the cap after use.

Repeat these steps every day

During the first month, apply the treatment twice per day. Thereafter, continue treatment by applying the lotion once per day for as long as the nail remains visibly affected. The treatment of nail fungus may take 3 to 6 months or longer.

To ensure effective daily treatment, consider planning your daily application at the same time or moment of the day, for example after taking a bath or shower or by setting a daily reminder on your mobile phone. To support the healing process, wearing comfortable and well-fitting shoes is advised.

Nailclin has been designed to contain enough liquid for a treatment of 1 nail for 6 months. When no more liquid appears when twisting the rotary end, the pen is empty. The device is not reusable and may be disposed of along with regular household waste when empty or no longer needed.

Warnings and precautions:

- Do not use if allergic or hypersensitive to any of the ingredients.
- For external use only; apply to the nails only.
- Do not apply if the infected nail is surrounded by cuts or lesions.
- In case of a hypersensitive or allergic reaction, pain, discomfort, or other undesirable effect, discontinue use immediately.
- Do not use on children below the age of 18 years.
- Do not use for treatment of multiple persons.
- In case of contact with the eyes, rinse thoroughly with water.
- In case of swallowing the product, consult a physician.
- If you wish to use this product during pregnancy or breastfeeding, first consult your general practitioner.
- In case of absence of improvement after 12 months, discontinue use.
- Store between 5° and 25° Celsius and out of sight and reach of children.
- Do not use after the printed expiry date, see carton box or pen label.
- Be sure to report any serious incidents that has occurred in relation to the device to the manufacturer (see carton box) and the competent authority on medical devices of your country (this may be the Ministry of Health).

What is nail fungus?

Nail fungus (onychomycosis) is a fungal infection of the finger- or toenail(s) causing white or yellow nail discolouration, nail thickening, and in some cases disintegration of the nail. Toenails are more often affected than are fingernails. It is the most common disease affecting nails and constitutes about a half of all nail afflictions. The infection is usually caused by a type of fungi called dermatophytes, most often by the species *T. rubrum*, among many others. Nail fungus can be a tenacious condition, often requiring long, persistent treatment to achieve full cure.

Manufacturer & Distributor:
Pronova Laboratories BV
Kruitpad 16
1398 CP Muiden
The Netherlands
www.nailclin.nl

Last revised: Oct 2023

CE 0481
NC16NL101